® *Cyclotest*

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Instructions for Use  
(7-11)

Mode d'emploi  
(12-17)

Modalità d'impiego  
(18-23)

**UEBE**<sup>®</sup>  
Germany

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# Instructions for Use

**In-vitro-diagnostic device for determination of pregnancy hormone hCG in the urine.**

Please read these instructions carefully before performing the test.

## General information:

Many changes in the female body are a crucial precondition for normal pregnancy and normal development of the fetus. The adaptation of the female body to pregnancy is mostly guided by hormones. Amongst the most important pregnancy hormones are oestrogen, progesterone and hCG (human chorionic gonadotropin).

The hCG level of healthy and non-pregnant women ranges from 0 to 10 IU/L (International Units per Liter). At the onset of a normal pregnancy, the hCG level increases within the first days of conception. Up until the 10th week of pregnancy hCG reaches a level of up to 230,000 IU/L. After delivery the hCG concentration reaches normal levels within a couple of weeks. This is why hCG is a very good marker for diagnosing pregnancy. The border level that confirms pregnancy is  $>10$  IU/L.

With this <sup>®</sup>Cyclotest pregnancy test, a test that has been used for years in clinics and surgeries in similar form, you have the opportunity to achieve certain knowledge. This pregnancy test tells you, whether you are pregnant, starting with a hCG concentration of 25 IU/L in your urine and is therefore a suitable test to be used as soon as your period is due.

## Helpful hints:

Principally you could perform this test any time day or night, but it is recommended to use the first urine of the day because the hormones are at their highest concentration at this time. If you want to do the test during the day, you should not go to the toilet for 2-3 hours before the test and not drink very much. In the case of a low hCG concentration (e.g. because the urine is highly diluted or the test is taken before your period is due or in the event of an ectopic pregnancy), the test may show a negative result, although you are pregnant.

Medication or diseases (i.e. trophoblastic diseases) that influence your hCG level, may also influence the test results. The normal antibody-pill, however does not interfere with the test results.

Due to the very high sensitivity of this test, very low concentrations of hCG are detected during the first days of pregnancy. But a considerable part of pregnancies result in miscarriage in the early stage of pregnancy (50% within the 1st to 4th week). Therefore seemingly false-positive results may occur.

## Performing the test (step by step)

### Test components:

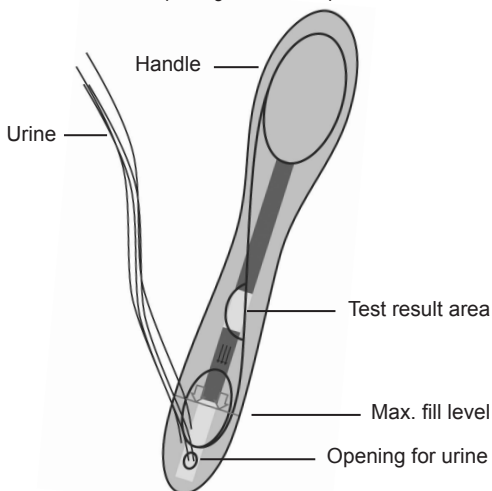
Box, instructions for use and foil pouch with test stick and desiccant.

In addition, you will need the following to perform this test: a watch with a second hand optional and optionally a clean container for your urine.

### Preparation:

Please note: Use the test immediately after opening the foil pouch.

Hold the test stick in a light stream of urine for approx. 3 seconds (see image). The amount needed runs down the stick into the opening for the sample.

If you prefer, you can collect the urine sample in a clean and dry container. Dip the opening for the urine sample into the collected urine for approx. 3 seconds.

**Important: Pay attention to the fill level!**

Lay down the test stick onto the foil pouch and wait 5 minutes.

## Test results

If 2 purple lines appear on the test stick, the test is positive. You can assume that you are pregnant also in case the lower line is extremely faint.

*Prenatal screening is very important. Please visit your doctor immediately to ensure for the health of both you and your baby. Please avoid potential risks for the embryo (e.g. X-ray).*

If there is only 1 purple line in the upper half of the result area, the test was performed correctly. You can assume that you are not pregnant.

*If your period does not start within a week of being due we recommend visiting your doctor. Please avoid potential risks during overdue period (e.g. X-ray).*

The test is invalid, if no purple line appears after 5-10 minutes. Reasons for an invalid result may be a damaged foil pouch, improper storage or a mistake when performing the test. In this case please keep all test components and contact the manufacturer of the test.

After 20 minutes the result can no longer be properly evaluated.

Dispose the test stick with the regular household waste.

PLEASE NOTE:

This is an immunological test for the detection of hCG (human Chorionic Gonadotropin) in urine. Although this test is extremely accurate, there may be inaccurate results in extremely rare cases. Components such as antibodies from animal sources may contain potentially infectious materials which are not dangerous provided the test has been performed in accordance with the directions.

## Q & A

*How does <sup>®</sup>Cyclotest pregnancy test work?*

<sup>®</sup>Cyclotest pregnancy test detects the concentration of the hormone hCG in the urine, which increases in case of pregnancy.

*How accurate is <sup>®</sup>Cyclotest pregnancy test?*

<sup>®</sup>Cyclotest pregnancy test is nearly 100% reliable (laboratory testing)

*When can <sup>®</sup>Cyclotest pregnancy test be done?*

<sup>®</sup>Cyclotest pregnancy test can be done from the first day of missed period.

*Is it a must to use the first urine of the day?*

No. The test can be done at any time of the day, but it is recommended to use the first urine of the day, because the hormones are at their highest concentration at that time.

*Can medication influence the result?*

Medication or diseases that influence the hCG level, may influence the test result. Pharmaceuticals like pain reliever or the normal anti-baby-pill does not interfere the result.

*How long is the test result readable?*

The result is shown for about 20 minutes. After that time the test area may change and a reliable reading is not ensured.

*What to do, if the test result is negativ and the period still missed?*

Often the cycle can change because of stress or discontinuing the pill. If the period does not appear during the next days, the test should be repeated. Is the result still negativ a doctor should be contacted.

## Please note

Read instructions carefully before use!

Use the test only once.

- Keep out of reach of children.
- Do not use the test after the expiration date and in case of damaged foil pouch.
- Store at 4° to 30°C (39° to 86°F).
- Please use the test immediately after opening the foil pouch.
- Don't take apart test stick.
- For external use only.
- In-vitro-diagnostic device for self-testing.
- This test conforms to Directive 98/79/EG of 1998 on in-vitro-diagnostics and bears the CE mark "CE 0483".

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UEBE Medical GmbH